- cially available water-repellent spray before first use. This protects against moisture.
- Repeat the process after several wears

**Cleaning**

- For light soiling, brushing the affected area is often sufficient. We recommend using a dirt brush for smooth leather and textile upper materials.
- A nubuck brush is required for suede and microfibre.
- Never clean the shoes in the washing machine

**Disinfection**

- The lining material can be cleaned with a damp cloth if necessary. We recommend using a disinfectant suitable for people with diabetes to clean the lining (these are available in pharmacies, for example).
- Do not use disinfectants on the upper leather. Do not wash LucRo special shoes in the washing machine.

**Leather care**

- Occasional scratches or insect bite scars in the leather are natural characteristics of genuine leather. To keep the leather supple for a long time, it must be nourished regularly. Suitable creams and nubuck care sticks are available from specialist retailers.
- Different leathers must always be treated with the appropriate shoe care products.

**Drying**

- Never dry wet or damp shoes near a direct heat source or in a tumble dryer.
- Avoid using radiators and direct sunlight to dry the shoes.
- If necessary, stuff wet or damp shoes with newspaper or use a suitable wooden shoe tree.
- Allow sufficient time for drying. Depending on the level of moisture, this may take up to two days.

**Storage**

- Store the shoes in a dry place at room temperature.

For detailed care and cleaning tips, please visit our website at [www.lucro.de/pflegtipps/](http://www.lucro.de/pflegtipps/)

**9. Disposal**

- The product and packaging must be disposed of in accordance with the statutory regulations.

**10. General information / Maintenance**

- Various factors influence the durability of the shoes.
- When used as intended and with proper handling and care, the product lifespan of the shoes is three years.
- Please observe our care instructions
- Check the shoes regularly for functionality, wear or damage (e.g. the fastener, sole, etc.)
- In the event of loss of function, wear and tear, or damage, you must stop wearing the shoes and consult your doctor or orthopaedic footwear technician.
- Arrange regular check-ups for your shoes and enter them in the table at the end of these instructions for use.
- We also recommend regular medical examinations of your feet, as appropriate to your condition (see recommendation below, Fig. 1).

**11. Assembly and installation instructions for specialist retailers:**

- Remove the shoe fillers (e.g. paper) and fit the shoes.
- Adjust the upper material if necessary. (e.g. correct width, hook-and-loop length)
- Sole reinforcement ONLY required for LucRo ergonomic mo-

dels: for people with diabetes, the reinforcement must be glued in place; for people with rheumatism, the use of the supplied reinforcement sole depends on the clinical condition. Please follow the assembly instructions carefully.

- The other product lines (classic, kinetic, sportic, urbanic, dynamic) are manufactured with torsion-free or low-torsion sole constructions to relieve the strain on the feet.
- For any modifications to the product, please note that the aid must be permanently labelled „custom-made“. From this point onward, the provider is considered the manufacturer of a custom-made device in accordance with MDR Annex XIII, and the CE mark becomes invalid.

**12. Warranty**

- We grant the statutory warranty for intended use of the product.

We recommend that you have your feet examined at regular intervals. The list of „National Care Guidelines“ serves as a reference.

Class	Findings	Examination	Risk
0	No sensory neuropathy	Annually	Low
1	No sensory neuropathy	Annually	Increased
2	Sensory neuropathy and signs of arterial occlusive disease and/or foot deformity	Every 3 months	Increased
3	Previous ulcer	Every 1-3 months	High

Figure 1: National Care Guideline Type 2 Diabetes: Prevention and treatment strategies for foot complications

Please inform Schein Orthopädie Service KG of any abnormalities, problems or deficiencies concerning the shoes. Due to regional legal regulations, you are required to immediately report any serious incident occurring with the use of this medical device to both the manufacturer and the authority responsible for your country.

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## Instructions for use

### For patients and orthopaedic specialists

\*This document is available in PDF format at www.schein.de for visually impaired persons.\*

Dear Customers, Thank you for your trust. With your purchase, you have made an excellent choice. In close cooperation with our partners, doctors, health insurance companies and medical services, it is our aim and commitment to ensure the best possible care of your foot with our shoes. Find out more about our products and visit our website:

**www.lucro.de**

**Please read these instructions for use carefully before using our medical product and observe the following notes, in particular the safety information:**

#### 1. Intended use / Indication:

- The special shoes are suitable for people with diabetes mellitus
- with pronounced neuropathy (loss of sensitivity),
  - with pronounced angiopathy (circulatory disorders),
  - with ulcers that have already healed and/or other mechanical irritations that lead to regional pressure peaks, or
  - for minimizing the risk of impending dorsal ulcers.

#### 2. Contraindications:

- Skin allergies to the materials used – please consult a doctor before use.
- Damaged skin or unhealed ulcers on the feet.
- Severe foot deformities.

**Please observe our instructions each time before wearing „LucRo shoes“, and especially before wearing them for the first time**

#### 3. Instructions for use:

Before the first use, the shoes for people with diabetes must be fitted by specialist staff according to the individual needs of the user. The specialist staff are responsible and liable for the correct fit of the shoes for people with diabetes.

The term specialist staff includes qualified medical device consultants or other persons who are authorised under national regulations to provide and/or supply medical devices.

**Specialist retailers** must fulfil the following requirements before and during product dispensing:

- Determine the correct length and width.
- Precise instruction on the correct use of the medical device (see patient obligations).

- Check the usability of the products, e.g. for function or damage.

#### 4. Patient obligations for the correct use of the medical device

- For your safety, put the shoe on while seated.
- Always check the inside of the shoes by hand before putting them on and remove any foreign objects.
- Always use a shoehorn when putting on the shoes
- If applicable, place the inserts/insoles in the correct shoe (left and right)
- Open and loosen the laces or fasteners before putting on the shoes. Step into the shoe and fasten it.
- Do not fasten the shoe too tightly to avoid pressure points and impaired circulation. Check the fastening immediately and adjust it if necessary. Fastening the shoe too loosely causes slipping inside the shoe, this must be

avoided.

- Do not pass the shoes on to third parties, as they are a medical device intended solely for one patient. Use by other persons is not permitted.
- Observe the daily duration and frequency of use as prescribed by your doctor.
- We recommend changing the shoes after one day of wear.
- Check your feet for pressure points and redness after each use.
- If you are unable to put on the shoes yourself, another person familiar with the instructions for use should assist you.

#### 5. Location / Application conditions:

- Use the street shoes only outdoors and the house shoes only indoors.
- Dry outdoor models after use in the rain (according to the care instructions).
- For hygienic reasons, we recommend three pairs of shoes: one for indoor use and two pairs for outdoor use.

**Please note that there is a risk of slipping on wet or slippery surfaces, as well as on snow and ice. Special caution is required in this case.**

#### 6. Safety instructions and (application) risks:

- Always check the inside of the shoes by hand before putting them on and remove any foreign objects.
- Never wear the shoes without a LucRo insole or a special insole provided by your orthopaedic footwear specialist.
- Only wear the shoes with socks (clean and intact). We recommend special socks for people with diabetes.
- At first, wear new shoes only for short periods and check your feet for pressure points after approx. 20 minutes.
- If pressure points, pain or other adverse symptoms occur, consult your treating physician or the orthopaedic footwear technician immediately.
- Shoes that are fastened too loosely can cause falls or pressure points due to slipping.
- Do not fasten the shoe too tightly to avoid pressure points and impaired circulation Check the fastening immediately and adjust it if necessary.
- You should examine your feet daily for redness and pressure points.

**Important note:** **If you have any problems with your feet or shoes, please contact your doctor or the specialist orthopaedic (shoe) retailer who supplies you immediately!**

#### 7. Material specifications:

Various **smooth** and **suede leathers** and **textiles** are used in LucRo shoes.

- Upper material:** Upper material – smooth leather, suede leather, as well as textiles (depending on the model, the textiles used may include nonwovens, mesh materials, canvas, microfibre, or felt)
- Lining material:** Microfibre
- Footbed:** Polyurethane
- Outsole:** Polyurethane, ethylene-vinyl acetate

The materials for the lining and footbed are the same for all models.

#### 8. Cleaning and care instructions

##### Before first use

- Spray the upper material of your shoes with a commer-